*No. 1 choice
of hospitals
& mothers*

Base

- EN** Instructions for use
- SV** Bruksanvisning
- DA** Brugsanvisning
- FI** Käyttöohjeet
- NO** Bruksanvisning
- PL** Instrukcja obsługi

EN Parts List XI

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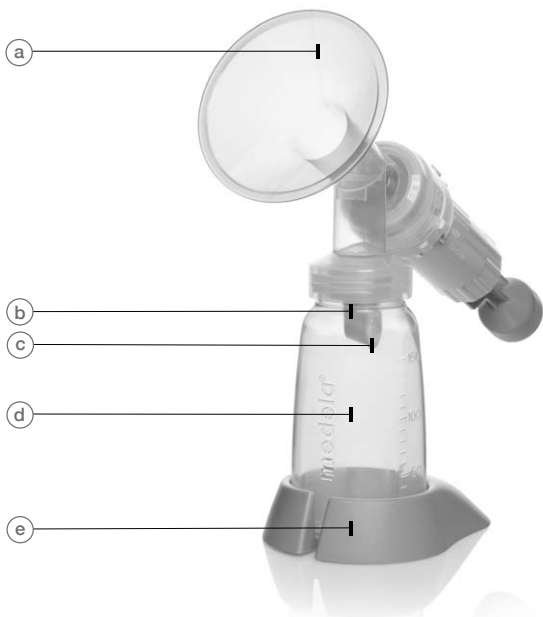
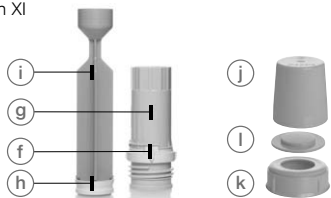
DA Reservedelsliste XI

FI Osaluettelo XI

NO Reservedelsliste XI

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EN	Instructions for use	4
SV	Bruksanvisning	12
DA	Brugsanvisning	20
FI	Käyttöohjeet	28
NO	Bruksanvisning	36
PL	Instrukcja obsługi	44

I Manual Breastpump

Mother's milk is the best nutrition for babies. The Manual Breastpump gives you and your baby optimal support during the breastfeeding period. The attractive manual breastpump is especially suitable for occasional pumping at home or on the go.

II Indications For Use

The Manual Breastpump is a breastpump:

- To be used by lactating women to express and collect milk from their breast during absences from the baby (e.g. an evening out from baby, working part-time).
- To relieve engorgement occasionally.
- For home use and on the go.

III Important Safeguards

Read all instructions before use.

- *Use the product only for its intended use as described in this instruction manual.*
- **Use Medela original accessories only.**
- *Check pump kit components for wear or damage.*
- *This is a single user product. Use by more than one person may present a health risk.*
- *If problems or pain occur, consult your lactation consultant or physician.*

SAVE THIS INSTRUCTION MANUAL

IV Assembly Instructions

1

Snap membrane onto valve until membrane lies completely flat.

b + c

2

Push assembled valve and membrane onto breastshield. Ensure side-wise position of valve as indicated on picture.

a1 + a2

b c

3

Screw bottle into breastshield.

a b c

+

d

4

Slip rubber seal onto piston until it snaps into place - the larger side faces the handle. To ensure proper seal, hold rubber seal firmly and twist piston.

h

+

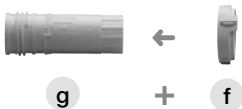
i

IV Assembly Instructions

5


Slide vacuum regulator ring onto cylinder.

6

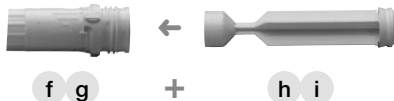
Slide piston into cylinder.

7

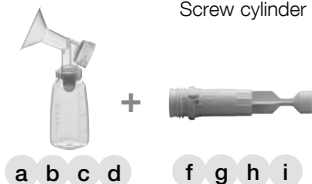Screw cylinder into breastshield.

V Pumping

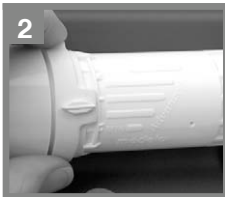
Be comfortably seated and relax. Use a footrest, when pumping or when feeding your baby. Elevating your feet puts you into proper sitting position to avoid stress on the lower back and perineum.

1

Center your breastshield over your nipple.

Do not tilt or overfill bottles when pumping.

2

Set the vacuum regulator to minimum and start pumping. Adjust the vacuum to your **Maximum Comfort Vacuum™**. This is the highest vacuum you can use and still feel comfortable (different for every mother). To find your Maximum Comfort Vacuum increase vacuum until pumping feels slightly uncomfortable (not painful), then decrease slightly.

Research has shown that you will pump more efficiently – meaning get more milk in less time – when pumping at Maximum Comfort Vacuum™.

Once finished pumping disassemble the pump kit items. Close the bottle and store it for later use.

Use bottle stand to prevent tip over.

VI Breastshield Fitting

Choosing the right size of Breastshield

A correctly fitted breastshield avoids compression of any milk ducts during pumping to ensure optimal breast emptying and maximum milk output.

Your breastshield is the right size if your answer to the following questions is «Yes»:

- Is your nipple moving freely in the tunnel?
- Is minimal or no areolar tissue being pulled into the tunnel of the breastshield?
- Do you see a gentle, rhythmical motion in the breast with each cycle of the pump?
- Do you feel the breast emptying all over?
- Is your nipple pain-free?

If your answer to any of the above questions is «No», try a larger (or smaller) breastshield.

Medela PersonalFit™ Breastshield: Available in five sizes – because one size does not fit all.

VII Cleaning and Hygiene

Follow these cleaning instructions unless you are told otherwise by your healthcare professional.

Important

- Use drinking water quality for any of the following described cleaning procedures.
- Clean parts immediately after use to avoid dry up of milk residues and to prevent from growth of bacteria.
- Disassemble all parts prior to cleaning. Especially be sure to remove the yellow valve head from the breastshield and separate the white valve membrane from the valve head.

**Prior to first
use and
once daily**

- Disassemble, wash and rinse all parts of your pump kit that come in contact with the breast and milk (a+b+c+d+j+k+l).
- Place parts in a saucepan of sufficient capacity.
- Fill with cold water until the parts are well covered. Optionally, a tea spoon of citric acid may be added to avoid a build-up of limescale.
- Heat it up and allow to boil for 5 minutes.
- Remove from stove and drain water off.
- Dry parts with a clean towel.

Alternatively, the disassembled parts can be sanitized in top rack of dishwasher or steam-sterilized.

Note: Food colorings may discolor components which will not affect performance.

**After each
use**

- Disassemble and handwash all parts of your pump kit that come in contact with the breast and milk (a+b+c+d+j+k+l) in warm, soapy water.
- Rinse in clear water.
- Dry parts with a clean towel.

Hints

After cleaning, when parts are not in use, put in a clean zip lock bag or store in a container with lid. Alternatively, the parts can be wrapped in a clean paper or cloth towel.

Hygiene

- Wash hands thoroughly with soap and water before touching the breast and pump parts.
- Avoid touching the inside of bottles and lids.
- Clean the breast and surrounding area with a damp cloth.
- Do not use soap or alcohol which can dry the skin and cause cracked nipples.

VIII Storing / Thawing Breastmilk

Check with your hospital for specific instructions.

Breastmilk Storage (For Healthy Term Babies)

	Room Temperature	Refrigerator	Refrigerator Freezer Compartment	Deep Freezer
Freshly expressed breastmilk	Do not leave at room temperature	3 - 5 days at approx. 4° C	6 months at approx. -16° C	12 months at approx. -18° C
Thawed breastmilk (previously frozen)	Do not store	10 hours	Never refreeze	

- When freezing do not fill bottles/bags more than 3/4 full to allow space for expansion.
- Label bottles/bags with the date of pumping.
- Do not store breastmilk in the door of the refrigerator.
- In order to preserve the components of the breastmilk, thaw in the refrigerator overnight or by holding the bottle/bag under running warm water (max. 37° C).
- Gently shake the bottle/bag to blend any fat that has separated.
- Do not thaw frozen breastmilk in a microwave oven or in a pan of boiling water (danger of burning).

IX Troubleshooting

If there is low or no suction

Make sure

- the breastshield forms a complete seal around the breast.
- all connections of the kit and breastpump are secure.
- the white valve membrane is laying flat against the yellow valve head. Check that the valve head and membrane are clean and not damaged. See «Assembly Instructions».
- the rubber seal is correctly attached onto piston.
- the piston/cylinder is correctly attached onto breastshield.

X Disposal

The unit is made of plastics and may only be disposed of in accordance with local regulations.

XI Parts List

Article numbers

a1	Breastshield (24 mm)	810.7084
a2	Connector	810.7088
b	Valve head	810.0475
c	Valve membrane	820.7013
d	Bottle (150 ml)	811.0020
e	Bottle stand	810.0462
f	Vacuum regulator ring	820.0066
g	Cylinder	810.0717
h	Rubber seal	820.0078
i	Piston	810.0494
j	Cap	810.0474
k	Lid	810.7062
l	Disc	810.0473

Does not contain Bisphenol-A, DEHP (Phthalate) or Latex.

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Medela AG
Medela Medical AB